



LiquiBand® Optima

The fast, precise and controlled topical skin closure.

Our winged applicator allows for **effortless activation** and **controlled application**




✓ UNIQUE APPLICATOR

- ▶ Winged applicator allows easy and safe priming
- ▶ Visidome reservoir provides visualization of remaining adhesive

✓ EFFECTIVE SCALP WOUND CLOSURE

- ▶ Safe, effective and rapid wound closure for scalp lacerations
- ▶ Precision tip ideal for areas of dense hair growth

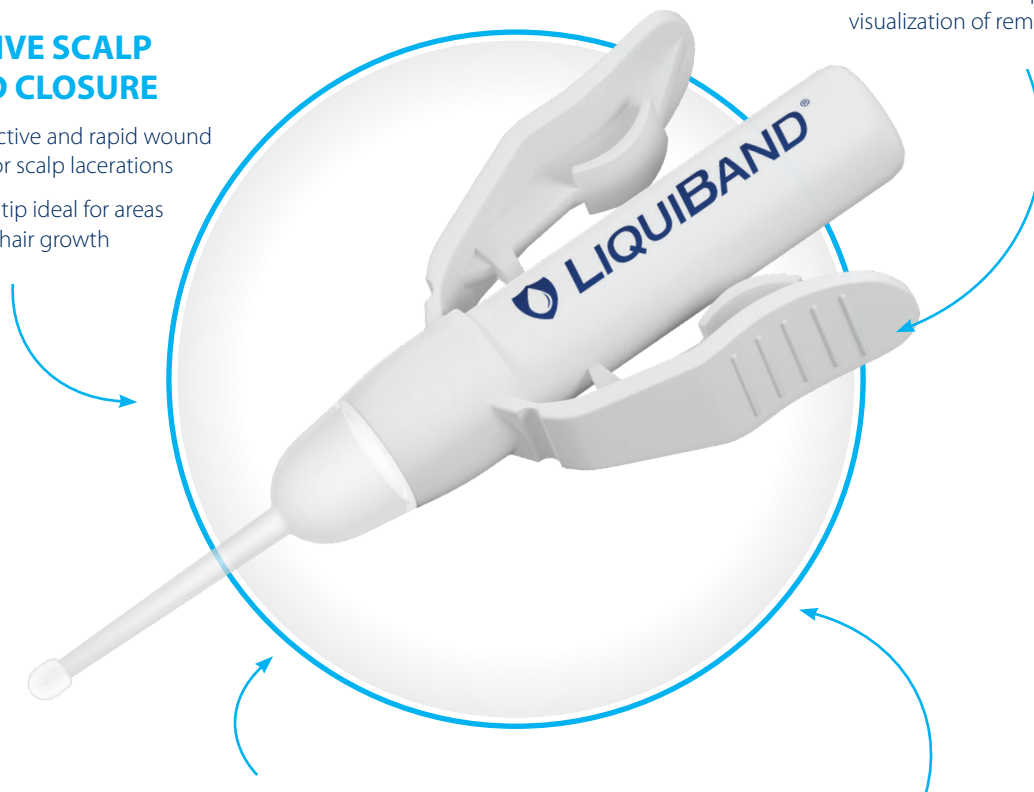


✓ ENHANCED PATIENT BENEFITS¹

- ▶ Offers excellent post-operative cosmesis
- ▶ Provides low exothermic reaction

✓ BUTYL-CYANOACRYLATE BASED FORMULA

- ▶ Strong and secure wound closure
- ▶ Quick 10-second dry time
- ▶ Controlled placement of adhesive

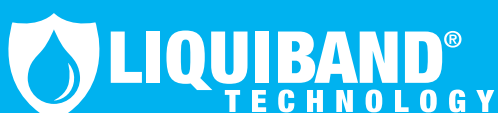

Better wound closure starts with a better tool

Application Guide **LiquiBand® Optima**

LiquiBand® Optima Topical Skin Adhesive has a unique, winged applicator to facilitate safe and easy activation, along with accuracy and control.

✓ STEP ONE

- ▶ Ensure that wound is thoroughly clean and dry prior to closure.
- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard.



✓ STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device. When the tip fills with adhesive, LiquiBand® Optima is ready to use.



✓ STEP THREE

- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Optima over the length of the wound.
- ▶ Continue to hold the wound edges together for 10 seconds.



LiquiBand® Optima

Size	Qty Per Box	Product Code	NHS SC Code
0.5g	10	OPT001	FVF 009

* If being used in areas of dense natural hair, take care when applying as hair follicle damage may occur if over applied.