

# ClearView®

## Uterine Manipulator



ClearView® Uterine Manipulator, with its 210° range of motion and rotating handle facilitates one-handed control, enabling the surgeon to manipulate the uterus independently.



### A UTERINE MANIPULATOR



#### Clear Visibility

ClearView provides 120° anteversion and 90° retroversion as well as lateral movement for clear visibility.



#### Single-Handed Control

The self-holding rotating control handle allows the surgeon to maneuver the device independent of an assistant.



#### Single-Use

ClearView is completely disposable.



#### Patient Positioning

ClearView pivots at the cervical os and allows the patient to remain in the supine position.

### PACKAGED WITH WHAT YOU NEED

ClearView is packaged in a sterile tray with everything you need to complete the procedure.

#### Chromotubation

The dye-injection port allows for chromotubation if needed.

#### Uterine Control

The included 12ml syringe inflates the tip balloon.



#### Disposable Sound-Dilator

The included sound-dilator helps measure and prepare the cervix.

#### Two Sizes

The spacer shortens the tip by 1-cm, giving you two sizes in one device.

## DOING A LAPAROSCOPIC HYSTERECTOMY?

# ClearView® Total

## Uterine Manipulator

ClearView® TOTAL has the same features as ClearView with the addition of a pre-attached occluder balloon and three different sized colpotomy cups. The occluder balloon prevents the loss of pneumoperitoneum once the colpotomy incision is made during laparoscopic hysterectomy.



4cm




3.5cm




3cm

## CLEARVIEW SET UP & USE

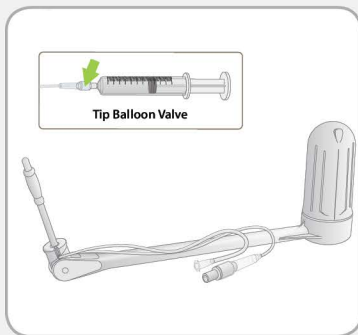


### Check Balloon

Use supplied syringe to test the balloon prior to insertion.

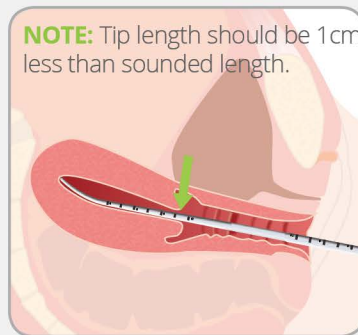


### Dilate & Sound Uterus

Sound the uterus. If necessary, dilate the cervix.

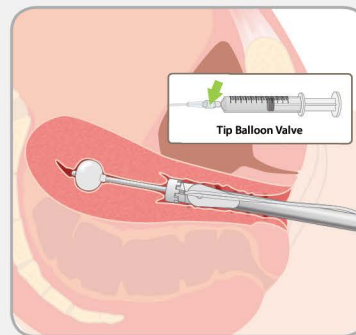

### Insert Device & Inflate

Lubricate distal tip and insert into uterus. Slowly inflate

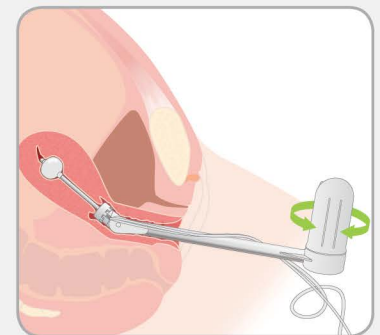

### Control Handle

Rotate the handle to manipulate the uterus

See Instructions for Use for full instructions, warnings, contraindications, and precautions.

## PRODUCT SUPPORT

### Ordering Information

Product Description	Product Code	Box Quantity
ClearView Uterine Manipulator Kit- 7 cm tip	UM-700	5/ea
ClearView Uterine Manipulator Kit- 9 cm tip	UM-900	5/ea
ClearView TOTAL Kit- 7 cm tip	UM-750	5/ea
ClearView TOTAL Kit- 9 cm tip	UM-950	5/ea
Disposable Sound- Dilator (Sterile)	DSD-1000	50/ea

Local Distributor:

[clinicalinnovations.com](http://clinicalinnovations.com)

747 West 4170 South | Murray, Utah USA 84123  
Phone +1.801.268.8200 | Fax +1.801.266.7373 |  
Toll Free +1.888.268.6222

