




1 Manual Retraction

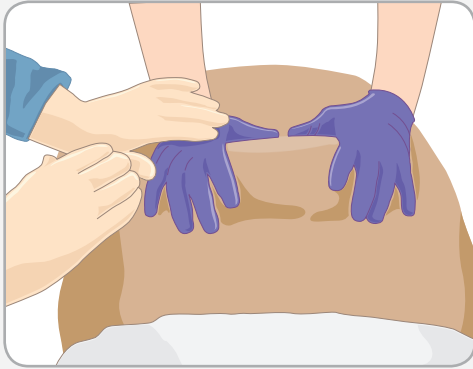


Manually retract panniculus to expose surgical site.

2 Remove 'A' Tab



Remove Tab 'A'.


3 Position & Apply

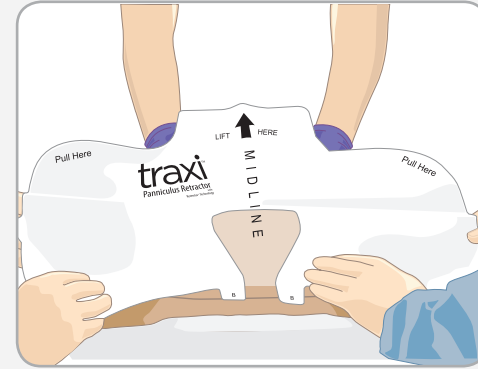

Position retractor 5cm above incision line.
Align to midline, apply to patient.


4 Remove 'B' Panels

NOTE: Backing removes by pulling down & away.

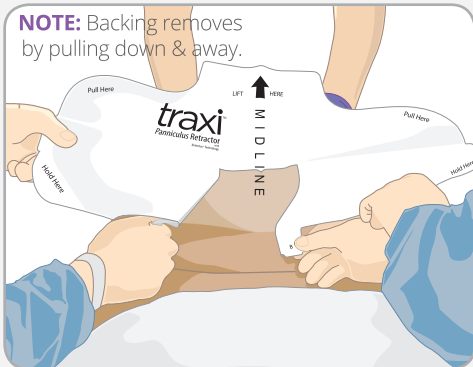


Remove the 'B' panels while holding film in tension above patient.

5 Tension & Apply

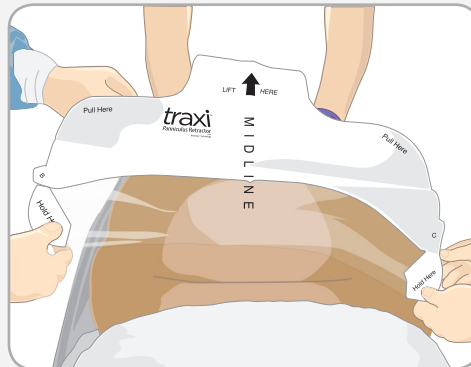

Hold in tension while smoothing down onto patient's skin, midline outward.

6 Remove 'C' Panel

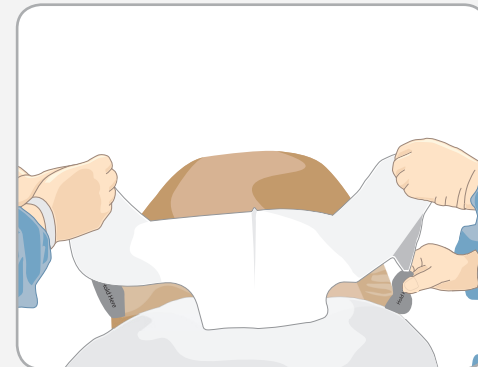

Retrograde device to relax panniculus to natural position. Remove 'C' panel by simultaneously pulling 'C' tabs.

7 Tension, Lift, Pull



Together, in tension, lift and pull retractor cephalad, anchoring at the xiphoid.



See Package Insert for Full Instructions, Warnings, Precautions & Contraindications

Quick Reference Guide- traxi Extender




1 Apply traxi




Apply traxi retractor. If suboptimal retraction is achieved, extender may be used.

2 Expand the Anchor Point

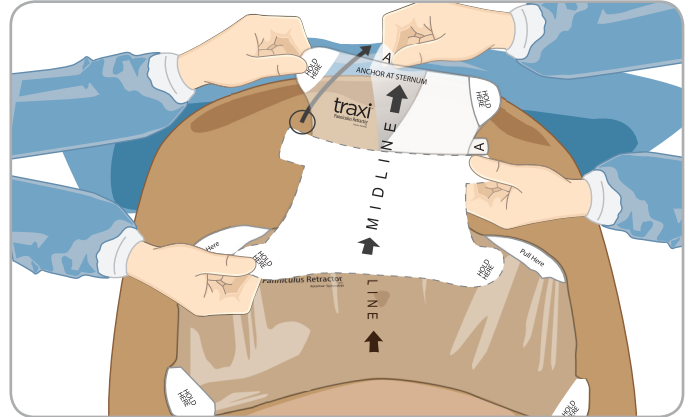


Remove 'A' panel by simultaneously pulling 'A' tabs from each side. Apply to sternum between breasts, near sternal notch midline — smooth onto skin.

3 Remove 'B' Panel

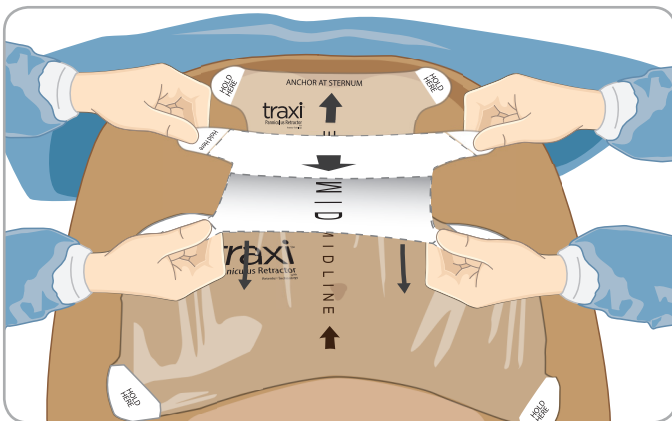

Remove 'B' panel by simultaneously pulling 'B' tabs from each side. Prepare to overlap and apply to traxi (do not apply yet).

4 Secondary Retraction

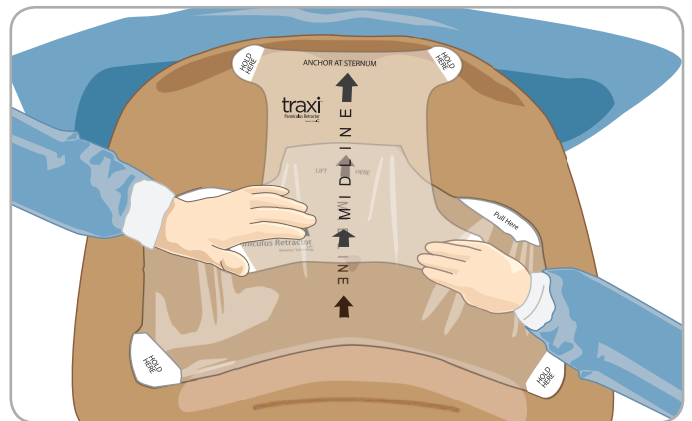

Before applying the extender, push the panniculus upward (toward patient's head). Holding the panniculus in place, smooth extender down onto the traxi toward umbilicus.



See Package Insert for Full Instructions, Warnings, Precautions & Contraindications

Local Distributor:

clinicalinnovations.com

747 West 4170 South | Murray, Utah USA 84123

Phone +1.801.268.8200 | Fax +1.801.266.7373 |

Toll Free +1.888.268.6222

© Clinical Innovations, LLC. All rights reserved. | P/N 050-0789 Rev. C

